

Promoting a lifetime  
of oral health starts  
with **SMART** choices.

The **SMART** choice  
in patient comfort.

## ULTRASONIC SCALING



### EFFICIENCY

Ultrasonic scaling can quickly and effectively remove even the most tenacious calculus, which means less chair time.<sup>1</sup>



### COMFORT

Patients experience less pressure and greater comfort during ultrasonic scaling procedures compared to hand scaling.<sup>2</sup>



### RE CARE

The more comfortable and less intimidating the procedure, the more likely the patient will return for more frequent dental visits.


**More dental professionals are turning to ultrasonic scaling.**

**SMART** tips to improve  
the patient experience.

## DURING SRP:



Maintain a light, feather-like touch similar to the exploratory stroke during ultrasonic scaling.

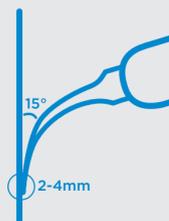

Adapt 2-4 mm of the active tip area and 0-15 degree angulation from long axis of tooth



Utilize short, overlapping strokes in a methodical pattern to debride the tooth surface.

To learn more about Smart Scaling Solutions, visit: [dentsplysirona.com/smartsaling](https://dentsplysirona.com/smartsaling)